



1 Prepare

Check donor ID
if necessary

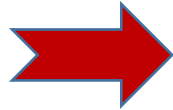


Explain to donor
the process of
testing and the
reason behind the
test

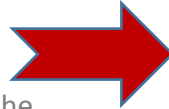

Complete Lab
Request Form
according to
instruction
(MATLFOR38)



The form is a Laboratory Request Form (MATLFOR38) from Matrix Diagnostics. It includes sections for Patient Information, Test Information, and a section for the donor to sign and date. There are also checkboxes for various test types and a section for the collector to sign and date.

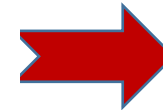


Record patient
information and collection
date on security seal and
prepare to take sample

2 Collect

Ask donor to
provide a
specimen into
Steralin tube
5ml minimum

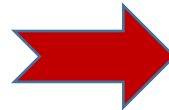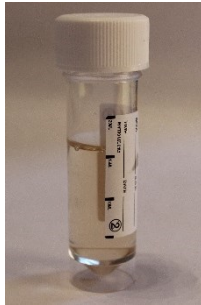

Apply barcoded
security seal from
Lab Request Form
over the top of
the steralin tube
and press down
firmly on both
sides.

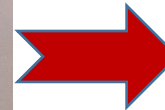


Place specimen tube
into transport tube
and screw on the
lid.



3 Seal & Send

Place transport tube and Lab Request
form into the Freepost Mail Bag

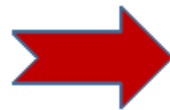

Send for testing via Royal Mail



Online Results are available via our portal
and through the Matrix App – please
contact your BDS for information

www.matrixdiagnostics.co.uk

labs@matrixdiagnostics.co.uk

01992 762 678